
Fiber Optic Otscope COMEDLIGHT

REF 24 850 40/ 41/ 45/ 47

We thank you for choosing our product. This equipment is primarily aimed to examining the ear canal and, more generally, to the lighting of surfaces as part of non-invasive tests.

Summary:

1. Safty in use
2. Setting up batteries / battery charge
3. Attaching a tip
4. Use
5. Magnifying Lens
6. General illumination
7. Insufflation port - pneumatic test
8. Changing the bulb
9. Cleaning and Sterilization

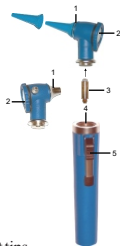1) Safety in use

Never insert the otoscope into the ear canal without tips. This appliance is intended for professional use, anyone not trained in its use is to prescribe.

2) Setting up batteries / battery charge

Unscrew the bottom of the handle of the otoscope and insert two alkaline batteries LR6AA in the battery cavity with the correct polarity (the « + » facing up).

3) Attaching a tip

The otoscope may only be inserted into the auditory canal when a disposable or reusable tip is fitted. Push the tip onto the instrument so that the internal projection engages in the slit (1). Twist to the right to lock.

4) Use

To turn on or off the otoscope push ahead or behind the black switch (4) on the top of the clip handle.

5) Magnifying lens

The lens in the viewing window (2) provides a magnification of about three times.

6) General illuminations

The otoscope can be used with or without a tip for general illumination of the oral cavity or skin. However avoid contact with mucous or wounds.

7) Insufflation port – pneumatic test

The otoscope Comedlight has a connection to eventually connect a blower. (Blower not supplied with the otoscope – not available by comed)

8) Changing the bulb

Unscrew the head of the otoscope and remove the bulb (3) by pulling it toward you. Insert the new bulb by pushing it to the limits of its receptacle.

9) Cleaning and sterilization

- Clean the outside of the otoscope head with a soft cloth or cloth moistened with a solution of alcohol 70% isopropyl or slightly soapy water. To clean the inside of the head, use a dry cotton or swab dampened with a little alcohol.
- The disposable tips should never be reused. Cleaning may damage the end and thereby injure the patient.
- Reusable tips can be cleaned with a disinfectant solution.
- Keep the otoscope away from heat and cold, oil and solvents.
- Avoid exposing the otoscope at sunlight.

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**REP** Obelis S.A Bd. General Wahis, 53 1030 Brussels, Belgium
**Medical Devices (Pvt.) Ltd** Wazirabad Road, Ugoki, Sialkot - Pakistan
COMED SAS - Parc des Forges - 8 rue Louise michel - 67200 STRASBOURG - France